



Cleaning Solutions

Walker Electronics Limited research over the past 40 years has shown that an Ultrasonic Cleaner will only work effectively when the fluid used is capable of dissolving or softening the contamination to be removed. An incorrect solution will slow down the cleaning process causing poor results and even damaging the Ultrasonic Cleaner.

The solution for your individual requirement can be selected from the list below, however, should you have any problems please call us and our trained staff will be pleased to advise you further.

RD3

A spirit based fluid for the effective cleaning and lubrication of hand pieces. It is essential to check with the hand piece manufacturer before use. Supplied in 5 litres. Use from container.

RD5

A Concentrated biodegradable non-ammoniated cleaning fluid specifically designed to remove a wide range of contaminations from instruments, burs, reamers, etc. Supplied in 5 litres.

Excellent Value - Dilutes to make 50 litres in total.

RD6

A high power ammoniated cleaning fluid which cleans and brightens a wide range of instruments. DO NOT use on Aluminium. Supplied in 2 litres. Dilute with water to make 5 litres.

RD7

To remove light deposits of tartar and calculus from dentures and trays. DO NOT use on aluminium and **ONLY** in a Glass Beaker. Supplied in 5 litres. Use from container.

DPR

Designed to remove dental plaster from dentures and trays. DO NOT use on aluminium. Supplied in 2 litres. Dilutes with water to make 5 litres in total.

Disinfectant WELclean

Our best selling cleaning fluid. It combines detergents and brightening agents with a disinfectant and is suitable for most instruments. Effective against Pseudomonas, Escherichia, Staphylococcus and Streptococcus. Supplied in 6x250ml or 2 litres. Dilute 1 part concentrate to 11 parts water.